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IFU-PTF-rev09/10.11.2022

TEFLON PLEDGET  Instruction for Use
Surgical, Polytetrafluoroethylene (PTFE) Felt
Nonabsorbable, Non-woven, White

BOZ TIBBİ MALZEME SANAYİ VE TİCARET A.Ş.
Saglık Mahallesi Sağlık 1 Sk. No: 33/5 Çankaya/Ankara/TÜRKİYE
Tel: +90 (312) 254 03 40 (5 hat) Faks:+90 (312) 254 03 50
web: www.boztibbi.com e-mail: boz@boztibbi.com

DESCRIPTION

Boz Pledgets are precut pieces from a sheet of polytetrafluoroethylene (PTFE) fabric that are used to spread the load of the suture against host tissue.

INDICATIONS

Boz Pledgets are indicated as sterile, single use, implantable devices placed in the heart during septal defect repairs, for patch grafting, to repair tissue, and to buttress sutures in cardiovascular surgery.

CONTRAINDICATIONS

There are no known contraindications.

WARNINGS

Users should be familiar with surgical procedures and techniques involving pledgets, as the risk of wound dehiscence may vary with the site of application.

Do not re-sterilize. Discard opened packages and unused pledgets. Acceptable surgical practice should be followed for the management of contaminated or infected wounds.

PRECAUTIONS

In handling pledgets, care should be taken to avoid damage from handling. Avoid crushing or crimping damage due to application of surgical instruments such as forceps or needle holders. The pledget, being implantable, should not be used with absorbable sutures.

Dispose of contaminated and unused products are in accordance with local and facility requirements.

SYMBOLS USED ON LABELLING

⊗ Do not reuse!
⊗ Do not resterilize
⊗ Do not use if package is damaged
🏭 Manufacturer
📅 YYYYY Date of Manufacture, Year
🕒 YYYYY-MM Expiry Date, Year - Month
STERILE EO Sterile EO: Ethylene Oxide
LOT Batch Number
CE 2292

REF Catalogue Number
°C 25 Store below 25°C
☀️ Keep away from sunlight
☔ Protect from humidity
♻️ Recyclable pack
⚠️ Attention, See instruction for use
📄 Consult instructions for use

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